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MT4104

### Wstęp

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**MT4104**

### Introduction

Congratulations on purchasing a smart otoscope 2 in 1 SMART OTOGUARD MT4104. The device will allow you to safely look inside the ears and nose and take care of their hygiene, with included medical quality head spoons.

Stop using dangerous cosmetic sticks, control the hygiene process yourself by gentle observing activities on your smartphone screen with a simple to use and intuitive app.

### Important

- Do not use the device in a place where you can be hit by other people
- The product is not intended to test kids under 3 years of age
- The product cannot be used by kids without supervision
- When washing the metal head of the otoscope avoid soak the rest of the device, containing battery and electronics
- Do not leave the device in full sun, as it is not resistant to UV rays and high temperature
- During normal use, the temperature of the device will rise to about 35°C which is a normal symptom.
- The device is designed to facilitate self-control for the user. However, be very careful when you are testing the other person.
- For better hygiene, replace head spoon with new one after up to 6 months of use.

### Kit Included

1. SMART DEVICE OTOGUARD MT4104
2. 4x transparent head spoons for removing deposits
3. 1x patented flexible silicone head spoon for removing deposits
4. Micro USB cable for charging the device's battery
5. This manual

### Device

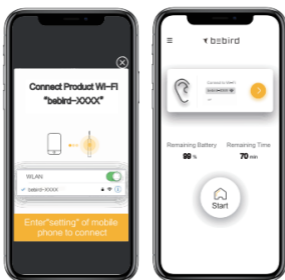
### Usage

Download from Google Play (Android) or Apple Store (IOS) app „bebird“.  
You can use the QR code below for this purpose

1. Install application „bebird“
2. Start the device by pressing and holding the ON/OFF button (5) to light up the LED in the head
3. Enable the smartphone's WIFI and connect to the network „bebird-XXXX“.


- If you receive the message „This network does not have internet access. Still want to connect to it?“, check the box „Don't ask for this network again“ and confirm „YES“
- Run the „bebird“ app. You will be asked to grant a number of consents to access your smartphone accessories (e.g. camera)

In most cases, an app user interface will appear on the screen along with a view from the otoscope camera. However, for a menu with a device selection, select the C3 model.

## Application

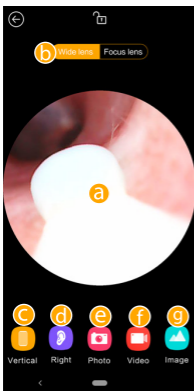- Otoscope camera preview
- Picture focus
- View Horizontal/vertical arrangement of the application window
- Determination of the left or right ear watching
- Taking a photo of the current image
- Recording of the video

**Important.** For technological reasons, photos and videos are saved at a lower resolution and with lower frames per second than the real-time image displayed on the smartphone screen.

- An album of saved photos and videos taken with your device. A short press allows you to preview, delete, or share image/movie

## Battery charging

The device is equipped with a lithium-ion battery. It should be charged at least every 30 days and not allow deep discharge. When the device is charging, the information light burns red. The LED will turn off when the otoscope is fully charged.

Due to continuous development, the specifications and appearance of the device are subject to change without prior notice. The current version of the user manual and technical support is available on [www.media-tech.eu](http://www.media-tech.eu).

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**GB Note on environmental protection:**  
After the implementation of the European Directive 2012/19/EU in the national legal system, the following applies: Electrical and electronic devices may not be disposed of with domestic waste. Consumers are obliged by law to return electrical and electronic devices at the end of their service lives to the public collecting points set up for this purpose or point of sale. Details to this are defined by the national law of the respective country. This symbol on the product, the instruction manual or the package indicates that a product is subject to these regulations. By recycling, reusing the materials or other forms of reworking old devices, you are making an important contribution to protecting our environment.

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