ACTIVEBAND TEMPERATURE EN

User manual

MT866

Introduction

Activeband TEMPERATURE MT866 is designed to be worn on dominant wrist and connects to Android or iOS device to help you monitor significant health parameters like Blood Pressure (BP), Heart Rate (HR), Blood Oxygen (O2), Body Temperature. It also keep track of your daily activities like quantity of steps you've taken, length of sleep and much more.

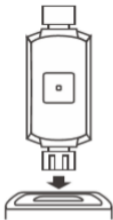
Device has a vibrate function which notifies you of alarms as well as incoming messages and calls. Device can be used to control camera on Android device. Vibrate alarm will prevent to lost your smartphone and also there is function which help to find connected smartphone by releasing ringtone.

Activeband TEMPERATURE requires Android system 4.4 or above or iOS 8.2 or above. Make sure that your mobile device supports the Bluetooth 4.0 low energy standard (BLE).

Introduction

First remove the wrist
band
Take out the fuselage

Plug into the USB charging head to charge

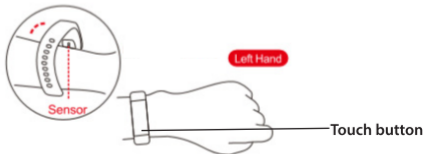
For the first time, please take out the bracelet and charge the device with the corresponding charger. When charging, the bracelet will start up automatically (charging mode is shown in the figure below: remove the bracelet body from the wristband and plug the charging port into the standard USB port to charge.)

Note.

Fast chargers is not supported.



Using device

Wrap device around your wrist medium tightly, about 3 cm from the wrist, to stick the sensor to skin surface.




Turn on the device by touching mode button under the screen. Using the same button you can switch modes.

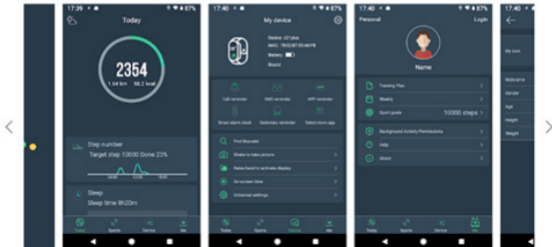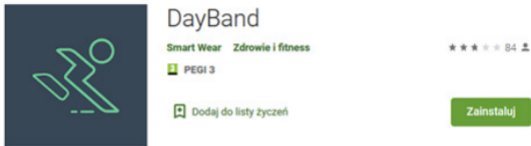To make initial setting (time, date, etc.) you have to install the DayBand app form Android Google Play store or App Store for iOS

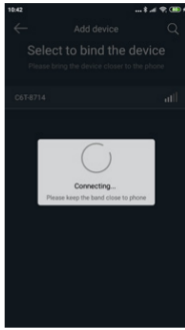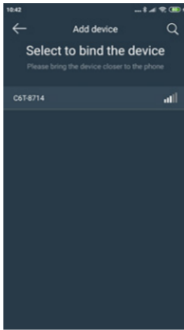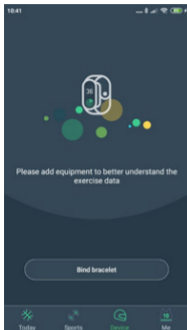Turn on Bluetooth and GPS interface in smartphone and run app. Go to "Device" tab and then click "Bind bracelet". Click item on the list to bind device.

Available modes

1. Time mode: display time, battery and date.
2. Step mode: display automatically recorded number of steps, calories burnt, calculated distance.
3. Heart rate monitor: automatically measure current heart rate (start displaying measurement after 10 seconds and shows averaged result after 60 seconds confirmed with vibration)

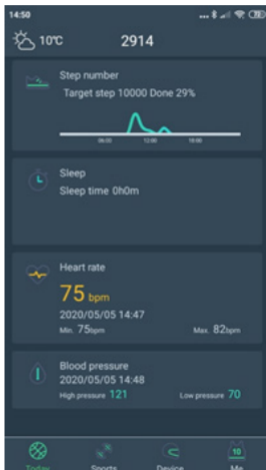
4. Blood pressure monitor: automatically measure diastolic and systolic blood pressure. Measurement process takes up to 150 seconds and finishes with vibration.
5. Blood oxygen monitor: automatically measure current quantity of oxygen and shows in %
6. Training mode: long press to enter running/cycling/skipping/badminton/table tennis.
7. Message: view notifications.
8. Centigrade thermometer: after few seconds display body temperature.
9. More: long press to enter options like info/find/brightness/shutdown/reset.

Note.


Distance between band and skin, dirt or sweat on sensor, heavy hair can influence the measurement quality. Please be noted that blood pressure monitor in the device will not replace the medical equipment and cannot be equal with professional measuring device

DayBand app

When device is linked to DayBand app it provides a lot of data about your daily, weekly and monthly activities. Also shows history of monitored health parameters.

DayBand app

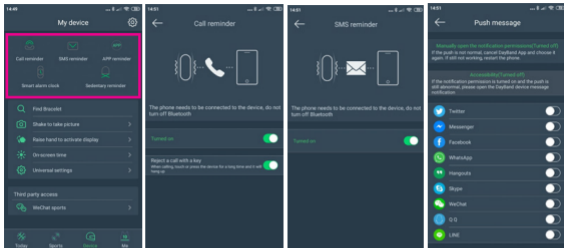If you would like to have notification about incoming calls/SMS/etc. please make sure that option is enabled as shown. Also please check if DayBand app has permission to access this function in smartphone (go to Android settings/app/DayBand/permissions).

Other options

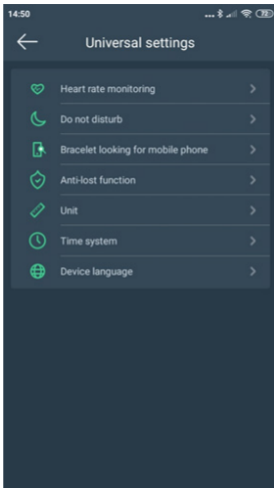
Find Bracelet – send request to band and make it vibrate.

Shake to take a picture – open camera module.

Raise hand to activate display – enable option to activate display by raising hand.

On-screen time – 5/10/15 seconds

Universal settings – show universal setting menu.

Due to continuous development specification and appearance of product are subject to change without prior notice. For technical support please visit www.media-tech.eu.

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
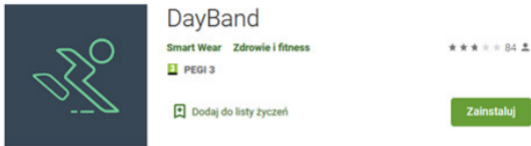
The image shows the Google Play Store listing for the DayBand app. On the left is the app icon, a green line-art figure of a person running on a dark blue background. To the right of the icon, the app name "DayBand" is displayed in a large, white, sans-serif font. Below the name, the categories "Smart Wear" and "Zdrowie i fitness" are listed in green. A green badge with a white "1" and the text "PEGI 3" is positioned below the categories. To the right of the app name, there are five stars and the number "84" with a person icon, indicating the app's rating. At the bottom right of the listing is a prominent green button with the white text "Zainstaluj". Below the app name, there is a link that says "Dodaj do listy życzeń" with a bookmark icon.

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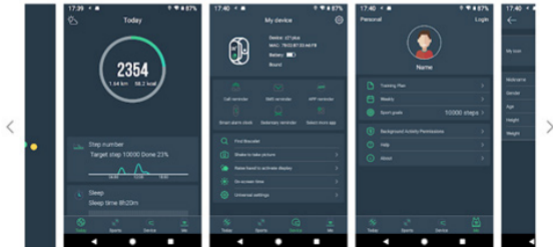GB Note on environmental protection:

After the implementation of the European Directive 2012/19/UE in the national legal system, the following applies: Electrical and electronic devices may not be disposed of with domestic waste. Consumers are obliged by law to return electrical and electronic devices at the end of their service lives to the public collection points set up for this purpose or point of sale. Details to this are defined by the national law of the respective country. This symbol on the product, the instruction manual or the packaging indicates that a product is subject to these regulations. By recycling, reusing the materials or other forms of utilising old devices, you are making an important contribution to protecting our environment.

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